

Concept of customized Bonegraft material



Sticky Bone without Blood Collection!


Bovine Bone Mineral

- Unable to mold
Lack of retentivity
- Scattered & Flowing
- Blood collection to make
sticky bone




- Moldable & Customizing
- Customized build up
- Sticky bone made easily
with saline or blood

BOVINE BONEGRAFT

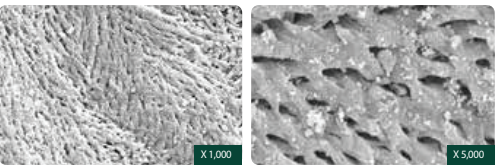

S1 has similar structural characteristics with human bones

Due to its high porosity, S1 provides the most optimal environment for promoting osteoblast adhesion, blood vessel formation, and bone regeneration.

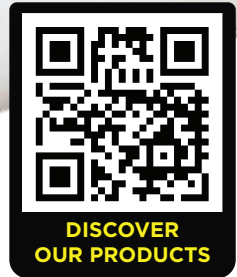

CUI: RO39133525, J22/817/2018
Șos. Ștefan cel Mare și Sfânt nr. 4
bl. A4, cam. 7 - 8, Iași
Website: pcdental.ro
Email: office@pcdental.ro
Tel: +40 (723) 219 818 | +40 (725) 092 000



NEW BONE
NEW LIFE

DENTAL
NATURAL BONE SUBSTITUTE



Keywords

NEW BONE NEW LIFE



S1 (Bovine Bone)

Bovine Bone Mineral


- 1 Excellent adhesion and fixation
- 1 Rigid Space Maintenance
- 1 Excellent osteoconduction



S1 (Bovine Bone)

- 1 Easy to manipulate
- 1 Various indications

High porosity

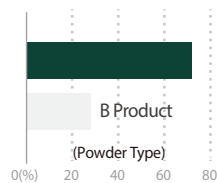

Porosimeter Test

- Percentage of porosity is more than 70%
- High porosity allows rapid penetration of growth factors


Improved Hydrophilicity



Hydrophilicity Test

- Porous structure makes blood circulation easily


Excellent shape maintenance

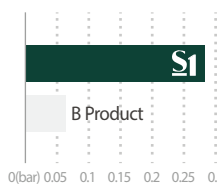

Pressure Test

- Perfect shape maintenance against high pressure from outside compared to other bone

Bone formation rate

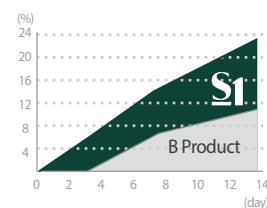

Mass Increase Test

- High attachment rate of ions to S1
- 21% increase in mass after 14 days due to high adhesive rate of ions





Quick Guide

✓ Please apply the liquid as indicated

✓ Each package is for one-time use only





✓ Do not mix with other bonegraft

✓ Mix with solutions well enough

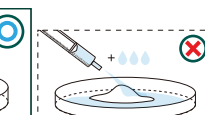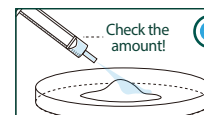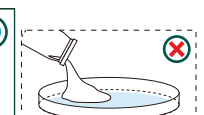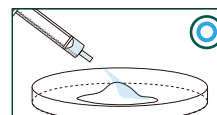

1 Place S1 in the tray and hydrate the materials with saline

1 Please use the recommended amount of saline only.

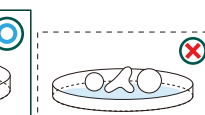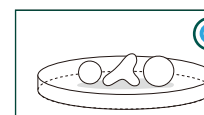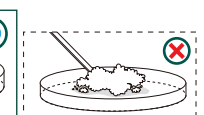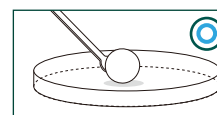

1 Knead the dough enough for at least 30 seconds by using hands or tools to form a lumpy shape before using S1.

1 Do not soak in saline after shaping for a surgery

Recommended Amount of Saline

TYPE	PARTICLE SIZE	WEIGHT	SOLUTION AMOUNT
Powder	0.2~1.0 mm	0.15 g	0.2 cc
		0.25 g	0.35 cc
		0.5 g	0.7 cc
		1.0 g	1.4 cc
		2.0 g	2.8 cc
TYPE	PARTICLE SIZE	WEIGHT	SOLUTION AMOUNT
Chip	1.0~2.0 mm	0.25 g	0.45 cc
		0.5 g	0.9 cc
		1.0 g	1.8 cc
		2.0 g	3.6 cc

* Ordermade : Medium Type (0.5~1.4 mm)

Innovative technology

Check it out for yourself!




Watch the clinical uses on our Youtube

Biopsy

※ Hematoxylin & Eosin Stain

Case 1 | #14, Ridge augmentation

Patient Info : 65 years-old, female

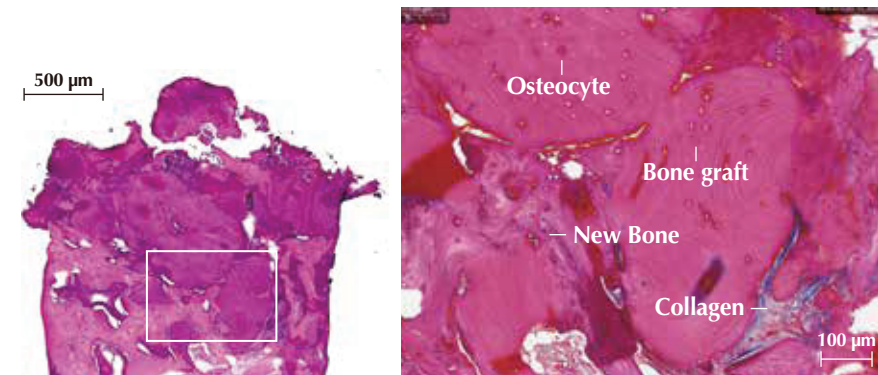


Clinical Opinion

- Excellent bone formation around bone graft material
- The graft materials appeared surrounded by newly formed bone
- No sign of inflammation or immune rejections

Case 2 | #26, Sinus graft

Patient Info : 73 years-old, female

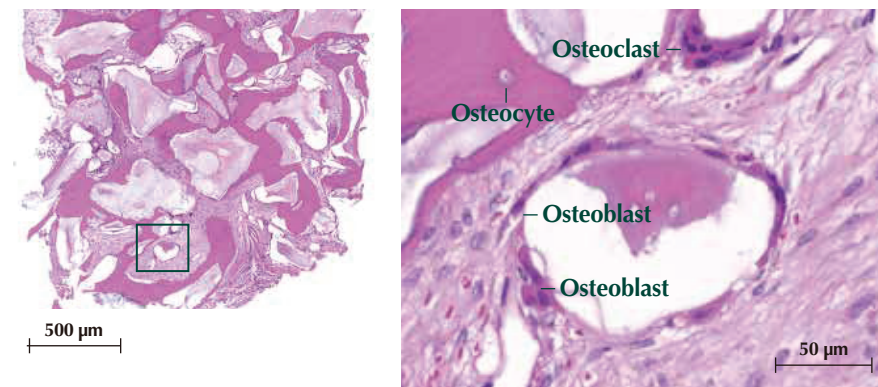

Clinical Opinion

- Excellent osteoconduction as it shows osteoclasts and new bone formation
- No evidence of inflammation or immune rejection

Clinical cases

Case 1

Patient Info : 59 years-old, female, Ridge augmentation



Case 2

Patient Info : 66 years-old, Male, Ridge augmentation

